



Initial Response, Inc.
Oxygen Bus Buddy Division
PO Box 525
Carrollton, VA 23314
FEIN# 54-2037122
877.377.4782 Fax: 757.357.9718
www.O2BusBuddy.com

O2 Bus Buddy Installation Sheet

**Thank you for safely transporting portable oxygen on school buses with the "O2 Bus Buddy Cabinet"!
This cabinet is designed to fit into most school buses with no modifications!**

PACKING LIST:

- 1 Installation Sheet
 - 1 Oxygen Bus Buddy Steel Cabinet with Oxygen Label
 - 2 "Quick Release Buckles" and Web Strap assembly
 - 1 Flame Retardant Padded Top Hat with mating Velcro®
-

INSTALLATION DIRECTIONS:

The "O2 Bus Buddy" is designed to mount to the existing seat rails on either side of a bus.

1. Remove the Tape backing of the Velcro strap from the Flame Retardant Padded Top Hat, and affix it to the top of the "O2 Bus Buddy" Cabinet.
2. Feed 1 "Quick Release Strap" through the bottom slot inside the cabinet. Feed the remaining "Quick Release Strap" through 1 of the 2 top remaining slots per height requirement of your Oxygen System.
3. Secure the "O2 Bus Buddy" Cabinet to the existing side, and floor seat rail using standard bolts that currently hold the seats to the rail. (If your bus does NOT have a floor rail, then mount it to the floor by drilling two holes through the floor and securing the unit on each side with 5/16" grade 8 bolts, washers, lock washers and nuts.) (Bolts Not Included).
4. Place the oxygen tank, or system into the "O2 Bus Buddy" between the cradles, and fasten it into place using the 2 "Quick Release Belts" provided. (Refer to Step# 3).
5. Adjust the Oxygen tubing so that it passes through the top 1" slot located at the top of the "O2 Bus Buddy" door.
6. Secure both latches of the door to the cabinet after confirming that the Oxygen system is secured.

Additional pictures and a copy of the Installation Sheet may be found online or by calling 877.377.4782

****Please note: The transportation of Oxygen should be done by trained professionals. The user of the "O2 Bus Buddy" should be completely familiar with the process of securing the "Quick Release Buckles", cabinet door, and completing a visual inspection of the cabinet, the mounting hardware, rails, the "Quick Release Buckles", the web straps, and the surrounding environment prior to EACH use of this product***

(This product, when installed must conform to any local, state, or federal regulations pertaining to the transporting of oxygen in your area.)

Thank You For Your Order

***Please report all Claims for Damages or Shortages within 10 Days After receipt of Merchandise.
All Returns require a Return Authorization (RA) Number.***