



Initial Response, Inc.
Oxygen Bus Buddy Division
PO Box 525
Carrollton, VA 23314
FEIN# 54-2037122
877.377.4782 Fax: 757.357.9718
www.O2BusBuddy.com

O2 Restraint Pouch Installation Sheet

Thank you for safely transporting portable oxygen with the "O2 Restraint Pouch"! This unit is designed for transportation vehicles such as small busses, vans, or even vehicle use. Our padded restraint pouch is for smaller portable oxygen systems to be secured for transport via existing three point seat belt systems! **PLEASE NOTE: The Oxygen Restraint pouch is NOT designed for "E" size cylinders!**

PACKING LIST:

- 1 Installation Sheet
- 1 Oxygen Restraint Pouch, IRP06



INSTALLATION DIRECTIONS:

The "O2 Restraint Pouch" is designed to secure individual portable oxygen cylinders size M-6, C, D, or other portable units with or without their existing bags, via three point seat belt systems. 5 of the 6 securing areas are to be used altogether for proper installation.

1. Place your portable oxygen unit system inside the Oxygen Restraint Pouch. The unit should be placed with the top (regulator and flow systems) corresponding with the top of the pouch. The "V" type peel is open at the top, and the bottom of the unit is closed with a circular bottom.
2. Secure the portable oxygen unit by **connecting the main buckle in the middle** and adjusting the tightness to the shape and size of the unit. Further **secure the buckle at the top**, gently closing the "V" peel gap to secure the unit, but never occlude the oxygen tubing.
3. This unit is conveniently designed for RIGHT or LEFT side seat loading. Connect your seat belt (shoulder-lap belt) system over the Restraint Pouch and then **connect "3" points of contact to the Restraint Pouch. One at the top (depending on the side) and two at the bottom.** This will properly connect the unit to your existing three point seat belt system.

**Please note: The transportation of Oxygen should be done by trained professionals. The user of the "Oxygen Restraint Pouch" should be completely familiar with the process of securing a three point seat safety belt system, completing a visual inspection of the Oxygen unit, Restraint Pouch, all buckles - closures, and the surrounding environment prior to EACH use of this product.*

(This product, when installed must conform to all local, state, or federal regulations pertaining to the transporting of oxygen in your area.)

Thank You for Your Order
Please report all Claims for Damages or Shortages within 10 Days After receipt of Merchandise.
All Returns require a Return Authorization (RA) Number.
877-377-4782